

AMD Packed Pixel Mode on Win7



To enable **AMD FirePro** cards to drive older Image Systems 3mp and 5mp Packed Pixel displays.
(Display serial # starts with PG21M.....or PN21M.....)

Use AMD driver 8.911.3.4

Open Catalyst Pro Control Center




Select "AMD FirePro™" and "AMD FirePro™ Settings"

Check box to "Enable 10-bit pixel format support"

Click "Apply"

Reboot computer

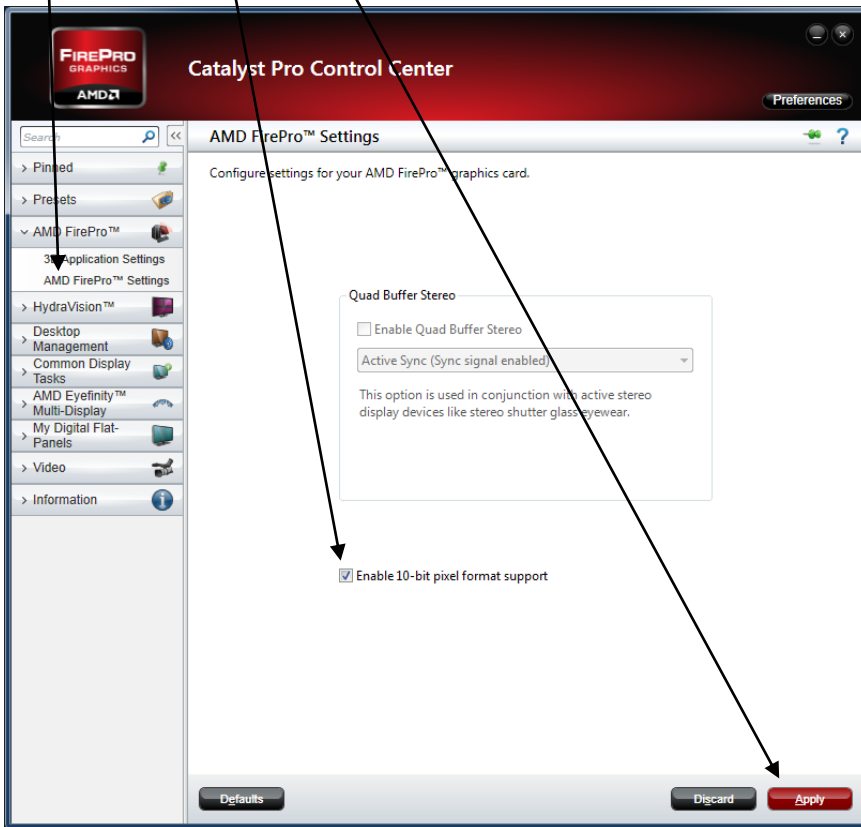

Adjust resolution to 1536x2048 (3mp) or 2048x2560 (5mp)

Reboot computer