

ATI Packed Pixel Mode

To enable **ATI FireGL** cards to drive older Image Systems 3mp and 5mp Packed Pixel displays.
(Display serial # starts with PG21M.....or PN21M.....)

Use ATI driver 8.773.1.1000

Open ATI Catalyst Control Center



Canvys Technical Support
800-235-2125



Select "Workstations"

Check box to "Enable 10-bit pixel format support"

Reboot computer

(Note: disabling this may fix resolution issues on Combo series)



Adjust resolution to 2048x1536 (3mp) or 2560x2048 (5mp)

Reboot computer

* If CFS is not able to detect the display/s, add A-B USB cables from each display to PC.