


AMD Packed Pixel Mode on WinXP



To enable **AMD FirePro** cards to drive older Image Systems 3mp and 5mp Packed Pixel displays.
(Display serial # starts with PG21M.....or PN21M.....)

Use AMD driver 8.911.3.4

Open Catalyst Pro Control Center

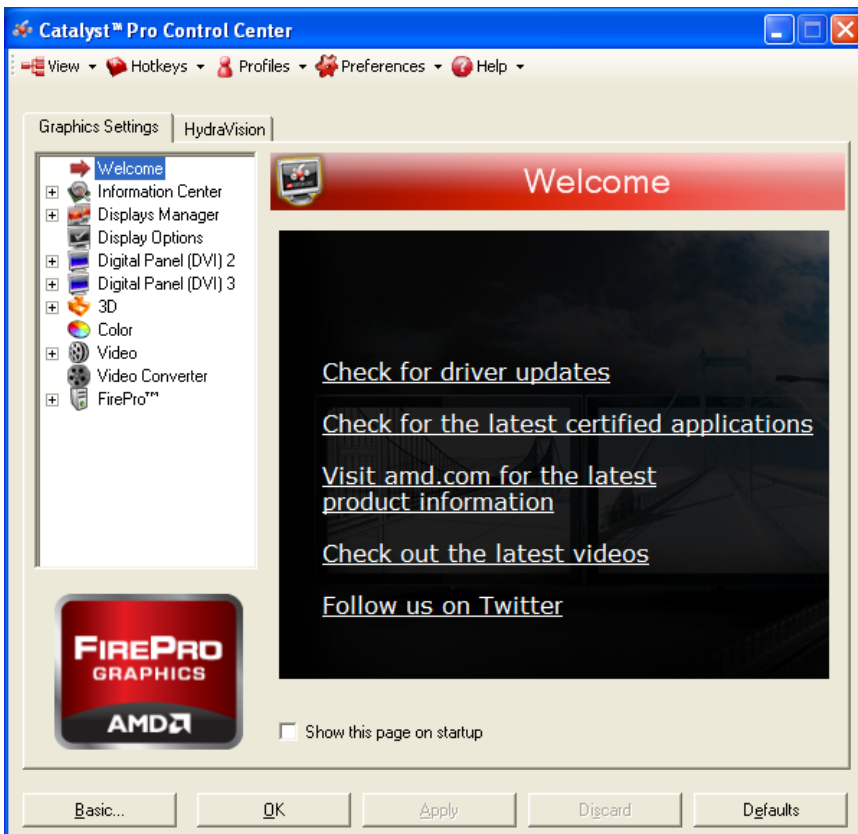


On the “Graphics Settings” tab, select “FirePro™” -> “Settings”

Check box to “Enable 10-bit pixel format support”

Click “Apply”

Reboot computer

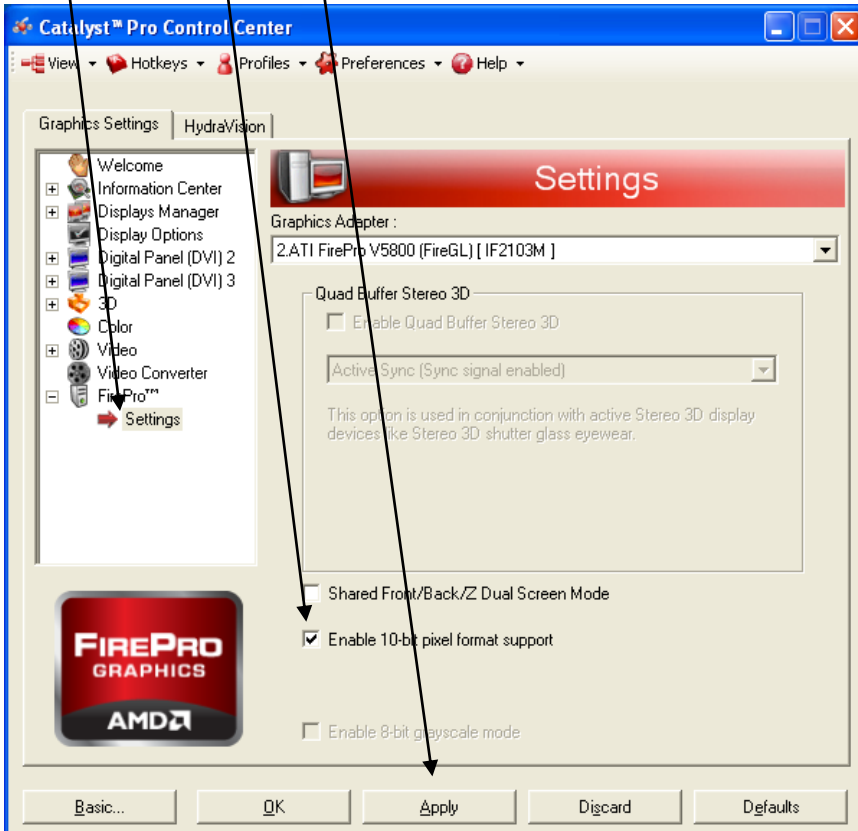

Go to “Displays Manager” -> “Displays Properties” to adjust the resolution (Desktop Area) to 1536x2048 (3mp) or 2048x2560 (5mp) and the rotation to Portrait (Left or Right 90°) and then click “Apply”.

Reboot computer